Sacrobelt

21030

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Intended use/Indications

Sacroiliac joint dysfunction and sacroiliac joint pain.

Contraindications

None known.

Adjustments

This product should only be altered by medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material specifications





Polyuretane, nylon, elastic.
Latex free.

Washing Instruction

Close all Velcro straps prior to laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

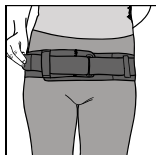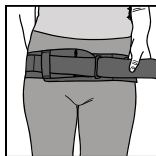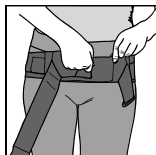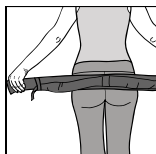Instructions

1. Check that the orthosis is complete and without damage.
2. Open the Velcro closure and the d-ring strap.
3. Position the belt low, over sacrum with the removable pad centered over the sacroiliac joint (the opening of the pouch pocket should be up).
4. Grasp both ends of the inner elastic band and stretch it straight out on both sides and pull. Attach it to the Velcro fastener. The belt should sit tight but comfortable.
5. Thread the outer nylon strap through the buckle, pull and attach. The outer band should provide extra support.
6. Check for comfort.

Med Spec Item no.	Size	CM
163302	Small	67-80 cm
163304	Medium	80-98 cm
163305	Large	98-116 cm
163306	X-Large	116-133 cm
163307	XX-Large	133-150 cm

Medical Specialties, Inc.
 4600-K Lebanon Rd.
 Charlotte, NC 28227
 p: 800-334-4143 f: 800-694-9060
 p: 704-694-2434 f: 704-694-9060
 e-mail: request@medspec.com
 www.medspec.com

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MDSS GmbH
Camp Scandinavia AB, Karbingatan 38